

## **IMPORTANT VOLUNTARY RECALL NOTIFICATION MST-7300 Micro Surgery Operating Room Table Series**

Department of Neurosurgery:

This notice is sent to you in accordance with the requirement of the U.S. Food and Drug Administration FDA. At Mizuho America, our first priority is to our customers and their patients, and that includes the safe and effective use of our products. Mizuho Corporation has initiated a voluntary product recall of MST-7300B and MST 7300BX Tables due to two types of errors.

- Unintended Tilting of the Tabletop (MST 7300BX) –
  - Impurities in the hydraulic oil has caused minor leaking inside a control valve causing the table to tilt after remaining static for long periods of time.
- Incorrect Operation due to Infrared Operating Signal Errors (MST 7300B & MST 7300BX)
  - The table can inadvertently mix infrared signals from the hand pendant while using in the wireless mode and may cause unintended movement.

This voluntary recall has been communicated to the U.S. Food and Drug Administration (FDA). We want to assure you that our parent company Mizuho Corporation is taking all necessary measures for a quick resolution to this Voluntary Recall.

As soon as possible one of our sales representatives will be contacting you to organize the logistics of the correction for these products. We recognize the correction may interrupt your schedule and to lower that interruption, we will schedule the correction to your best convenience.

You may also visit our webpage ([www.mizuho.com](http://www.mizuho.com)) where you can find additional information related to the Voluntary Recall. For additional questions, please contact our Customer Service Group or your local sales representative.

Please accept our sincerest apologies for the inconvenience and uncertainty, and trust that we are doing all we can. You have the commitment of Mizuho Corporation and Mizuho America to take the highest priority in resolving this issue and doing everything within our means to ensure the safety of your patients and peace of mind.

Once again, we are truly thankful for your loyalty, and appreciate your understanding as we work through this correction.

Sincerely



9/11/2019  
Dean Yamamura  
President & COO